

## Drugs of Abuse Assays Cross-Reactivity Manual by Generic Name, June 2005

COMPOUND (Generic Name)	COMPOUND (Trade Name)	RESULTS
Acetaminophen (See also Paracetamol)	Aceta	Non-reactive
Acetaminophen (See also Paracetamol)	Acephen	Non-reactive
Acetaminophen (See also Paracetamol)	Apacet	Non-reactive
Acetaminophen (See also Paracetamol)	Dapacen	Non-reactive
Acetaminophen (See also Paracetamol)	Feverall	Non-reactive
Acetaminophen (See also Paracetamol)	Tylenol	Non-reactive
Acetaminophen (See also Paracetamol)	Excedrin (combination)	Non-reactive
Acetaminophen (See also Paracetamol)	Panadol	Non-reactive
Acetaminophen (See also Paracetamol)	Tempra	Non-reactive
Acetophenetidin	Phenacetin	Non-reactive
Acetylsalicyclic acid	Aspirin	Non-reactive
Acetylsalicyclic acid	Anadin	Non-reactive
Acetylsalicyclic acid	Anasin	Non-reactive
Acetylsalicyclic acid	Bufferin	Non-reactive
Acetylsalicyclic acid	Caprin	Non-reactive
Acetylsalicyclic acid	Disprin	Non-reactive
Acetylsalicyclic acid	Ecotrin	Non-reactive
Acetylsalicyclic acid	Empirin	Non-reactive
Acetylsalicyclic acid	Excedrin (combination)	Non-reactive
Aluminum Chloride Hexahydrate	Drichlor	Non-reactive
Aluminum Chloride Hexahydrate	Anhydrol Forte	Non-reactive
Aluminum Hydroxide	Alu-Cap	Non-reactive
Aluminum Hydroxide	Alisone	Non-reactive
Aluminum Hydroxide	Gastrocote	Non-reactive
Aluminum Hydroxide	Kolanticon	Non-reactive
Aluminum Hydroxide	Maalox	Non-reactive
Aluminum Hydroxide	Maalox TC	Non-reactive
Aluminum Hydroxide	Mucogel	Non-reactive
Aluminum Hydroxide	Pyrogastrone	Non-reactive
Aluminum Hydroxide	Topal	Non-reactive
Alverine Citrate	Spasmonal	Non-reactive
Alverine Citrate	Spasmonal Fibre	Non-reactive
Aminopyrine		Non-reactive
Ammonia/Ipecuanha	Ipecac	Non-reactive
Amoxicillin or Amoxycillin	Amoxil	Non-reactive
Amoxicillin or Amoxycillin	Amoram	Non-reactive
Amoxicillin or Amoxycillin	Augmentin	Non-reactive
Amoxicillin or Amoxycillin	Heliclear	Non-reactive
Ampicillin	Penbritin	Non-reactive
Ampicillin	Polycillin	Non-reactive
Ampicillin	Principen	Non-reactive
Antazoline Sulphate	Otrivine-Anistin	Non-reactive
Aspirin	Bayer Aspirin	Non-reactive

<b>Aspirin</b>	Excedrin	Non-reactive
<b>Aspirin</b>	ASA	Non-reactive
<b>Aspirin</b>	Angettes	Non-reactive
<b>Aspirin</b>	Asasantin	Non-reactive
<b>Aspirin</b>	Caprin	Non-reactive
<b>Atenolol</b>	Beta-adlat	Non-reactive
<b>Atenolol</b>	Co-tenidone	Non-reactive
<b>Atenolol</b>	Kalten	Non-reactive
<b>Atenolol</b>	Tenben	Non-reactive
<b>Atenolol</b>	Tenif	Non-reactive
<b>Atenolol</b>	Tenoret 50	Non-reactive
<b>Atenolol</b>	Tenorectic	Non-reactive
<b>Atenolol</b>	Tenormin	Non-reactive
<b>Atorvastatin</b>	Lipitor	Non-reactive
<b>Azlocillin</b>	Securopen	Non-reactive
<b>Beclometasone</b>	AeroBec	Non-reactive
<b>Beclometasone</b>	AeroBec Forte	Non-reactive
<b>Beclometasone</b>	Asmabec	Non-reactive
<b>Beclometasone</b>	Beclazone	Non-reactive
<b>Beclometasone</b>	Becloforte	Non-reactive
<b>Beclometasone</b>	Becodisks	Non-reactive
<b>Beclometasone</b>	Beconase	Non-reactive
<b>Beclometasone</b>	Becotide	Non-reactive
<b>Beclometasone</b>	Fillair	Non-reactive
<b>Beclometasone</b>	Nasobec	Non-reactive
<b>Beclometasone</b>	Qvar	Non-reactive
<b>Beclometasone</b>	Ventide	Non-reactive
<b>Beclometasone</b>	Zonivent	Non-reactive
<b>Bendrofluazide</b>	Aprinox	Non-reactive
<b>Bendrofluazide</b>	Cogaretic	Non-reactive
<b>Bendrofluazide</b>	Indertic	Non-reactive
<b>Bendrofluazide</b>	Inderex	Non-reactive
<b>Bendrofluazide</b>	Neo-Naclax	Non-reactive
<b>Bendrofluazide</b>	Neo-Naclax-K	Non-reactive
<b>Bendrofluazide</b>	Prestim	Non-reactive
<b>Bendrofluazide</b>	Tenben	Non-reactive
<b>Benzalknonium</b>	Bradosol	Non-reactive
<b>Benzydamine</b>	Difflam	Non-reactive
<b>Benzylpenicillin</b>	Crystapen	Non-reactive
<b>Brompheniramine</b>	Dimetapp	Non-reactive
<b>Brompheniramine</b>	Dimotapp	Non-reactive
<b>Brompheniramine</b>	Dimotane	Non-reactive
<b>Bupropion</b>	Wellbutrin	Non-reactive
<b>Bupropion</b>	Zyban	Non-reactive
<b>Canestan</b>	Canestan ear/skin/vaginitis/HC	Non-reactive

<b>Carbamazepine</b>	Tegretol	Non-reactive
<b>Carbamazepine</b>	Teril	Non-reactive
<b>Carbamazepine</b>	Timonil	Non-reactive
<b>Cephalexin</b>	Ceporex	Non-reactive
<b>Cephalexin</b>	Keflex	Non-reactive
<b>Chloral Hydrate</b>	Welldorm	Non-reactive
<b>Chlorhexidine Gluconate</b>	Bactrigras	Non-reactive
<b>Chlorhexidine Gluconate</b>	Cordsodyl	Non-reactive
<b>Chlorhexidine Gluconate</b>	Chlorohex	Non-reactive
<b>Chlorhexidine Gluconate</b>	CX Powder	Non-reactive
<b>Chlorhexidine Gluconate</b>	Dermol	Non-reactive
<b>Chlorhexidine Gluconate</b>	Hibicet	Non-reactive
<b>Chlorhexidine Gluconate</b>	Hibiscrub	Non-reactive
<b>Chlorhexidine Gluconate</b>	Hibisol	Non-reactive
<b>Chlorhexidine Gluconate</b>	Hibitane	Non-reactive
<b>Chlorhexidine Gluconate</b>	Instillagel	Non-reactive
<b>Chlorhexidine Gluconate</b>	Naseptin	Non-reactive
<b>Chlorhexidine Gluconate</b>	Nystaform	Non-reactive
<b>Chlorhexidine Gluconate</b>	Serotulle	Non-reactive
<b>Chlorhexidine Gluconate</b>	Steripod	Non-reactive
<b>Chlorhexidine Gluconate</b>	Tisept	Non-reactive
<b>Chlorhexidine Gluconate</b>	Unisept	Non-reactive
<b>Chlorhexidine Gluconate</b>	Uriflex	Non-reactive
<b>Chlorhexidine Gluconate</b>	Uro-Trainer	Non-reactive
<b>Chlorpheniramine Maleate</b>	Galpseud	Non-reactive
<b>Chlorpheniramine Maleate</b>	Haymine	Non-reactive
<b>Chlorpheniramine Maleate</b>	Piriton	Non-reactive
<b>Chlorpromazine</b>	Largactil	Non-reactive
<b>Cimetidine</b>	Dyspamet	Non-reactive
<b>Cimetidine</b>	Tagamet	Non-reactive
<b>Cimetidine</b>	Zita	Non-reactive
<b>Citalopram</b>	Cipramil	Non-reactive
<b>Co-Amoxiclav</b>	Augmentin	Non-reactive
<b>Co-Amoxiclav</b>	Augmentin Duo	Non-reactive
<b>Co-Fluampicil</b>	Magnapen	Non-reactive
<b>Co-Phenotrope (atropine/diphenoxylate)</b>	Lomotil	Non-reactive
<b>Co-Phenotrope (atropine/diphenoxylate)</b>	Trepergen	Non-reactive
<b>Dexamethasone</b>	Decadron	Non-reactive
<b>Dexamethasone</b>	Dexa-Rhinaspray Duo	Non-reactive
<b>Dexamethasone</b>	Maxidex	Non-reactive
<b>Dexamethasone</b>	Maxidrol	Non-reactive
<b>Dexamethasone</b>	Minims	Non-reactive
<b>Dexamethasone</b>	Otomize	Non-reactive
<b>Dexamethasone</b>	Sofradex	Non-reactive
<b>Diclofenac Sodium</b>	Dicloflex	Non-reactive

Diclofenac Sodium	Diclomax	Non-reactive
Diclofenac Sodium	Motifene	Non-reactive
Diclofenac Sodium	Volraman	Non-reactive
Diclofenac Sodium	Volsaid	Non-reactive
Diclofenac Sodium	Voltarol	Non-reactive
Dicyclomine	Kolanticon	Non-reactive
Dicyclomine	Merbentyl	Non-reactive
Dimethicone	Asilone	Non-reactive
Dimethicone	Fancorsil	Non-reactive
Diphenhydramine	Benadryl	Non-reactive
Diphenhydramine	Medinex	Non-reactive
Diphenhydramine	Nytol	Non-reactive
Diphenhydramine	Panadol Night	Non-reactive
Dipenoxylate with Atropine	Lomotil	Non-reactive
Dothiepin	Prothiaden	Non-reactive
Doxycycline	Vibramycin	Non-reactive
Doxycycline	Vibramycin-D	Non-reactive
Erythromycin	Arpimycin	Non-reactive
Erythromycin	Benzymycin	Non-reactive
Erythromycin	Erycane	Non-reactive
Erythromycin	Erymax	Non-reactive
Erythromycin	Erythrocin	Non-reactive
Erythromycin	Erythroped	Non-reactive
Erythromycin	Ilosone	Non-reactive
Erythromycin	Isotrexin	Non-reactive
Erythromycin	Lactobinate	Non-reactive
Erythromycin	Stiemycin	Non-reactive
Erythromycin	Tiloryth	Non-reactive
Erythromycin	Zineryl	Non-reactive
Ethambutol	Myambutol	Non-reactive
Flucloxacillin	Co-fluampicil	Non-reactive
Flucloxacillin	Floxapen	Non-reactive
Flucloxacillin	Magnapen	Non-reactive
Fluoxetine	Prozac	Non-reactive
Fluoxetine	Sarafem	Non-reactive
Flupentixol Decanoate	Depixol	Non-reactive
Flupentixol Decanoate	Fluanxol	Non-reactive
Fusidic Acid (Sodium Fusidate)	Fucithalmic	Non-reactive
Fusidic Acid (Sodium Fusidate)	Fucibet	Non-reactive
Fusidic Acid (Sodium Fusidate)	Fucidin	Non-reactive
Gentamicin Sulphate	Cidomycin	Non-reactive
Gentamicin Sulphate	Genticin	Non-reactive
Gentamicin Sulphate	Gentisone	Non-reactive
Hydrocortisone	Actinac	Non-reactive
Hydrocortisone	Alphaderm	Non-reactive

<b>Hydrocortisone</b>	Alphosyl-HC	Non-reactive
<b>Hydrocortisone</b>	Anugesic-HC	Non-reactive
<b>Hydrocortisone</b>	Anusol-HC	Non-reactive
<b>Hydrocortisone</b>	Calmurid-HC	Non-reactive
<b>Hydrocortisone</b>	Canesten-HC	Non-reactive
<b>Hydrocortisone</b>	Colioam	Non-reactive
<b>Hydrocortisone</b>	Daktacort	Non-reactive
<b>Hydrocortisone</b>	Dioderm	Non-reactive
<b>Hydrocortisone</b>	Econacort	Non-reactive
<b>Hydrocortisone</b>	Efortelan	Non-reactive
<b>Hydrocortisone</b>	Efcortesel	Non-reactive
<b>Hydrocortisone</b>	Eurax-HC	Non-reactive
<b>Hydrocortisone</b>	Gregoderm	Non-reactive
<b>Hydrocortisone</b>	Hydrocortistab	Non-reactive
<b>Hydrocortisone</b>	Hydrocortone	Non-reactive
<b>Hydrocortisone</b>	Mildison lipocream	Non-reactive
<b>Hydrocortisone</b>	Neo-cortef	Non-reactive
<b>Hydrocortisone</b>	Nystaform	Non-reactive
<b>Hydrocortisone</b>	Perinal	Non-reactive
<b>Hydrocortisone</b>	Proctofoam	Non-reactive
<b>Hydrocortisone</b>	Quinocort	Non-reactive
<b>Hydrocortisone</b>	Terra-cortil	Non-reactive
<b>Hydrocortisone</b>	Timodine	Non-reactive
<b>Hydrocortisone</b>	Uniroid-HC	Non-reactive
<b>Hydrocortisone</b>	Vioform-HC	Non-reactive
<b>Hydrocortisone</b>	Xyloproct	Non-reactive
<b>Hydrocortisone-17-butyrate</b>	Locoid	Non-reactive
<b>Hydrocortisone-17-butyrate</b>	Locoid C	Non-reactive
<b>Hydroxychloroquine</b>	Plaquenil	Non-reactive
<b>Hydroxyzine</b>	Atarax	Non-reactive
<b>Hydroxyzine</b>	Ucerax	Non-reactive
<b>Hyoscine butylbromide</b>	Buscopan	Non-reactive
<b>Hyoscine butylbromide</b>	Scopoderm	Non-reactive
<b>Ibuprofen</b>	Brufen	Non-reactive
<b>Ibuprofen</b>	Codafen	Non-reactive
<b>Ibuprofen</b>	Fenbid	Non-reactive
<b>Ibuprofen</b>	Ibugel	Non-reactive
<b>Ibuprofen</b>	Ibuspray	Non-reactive
<b>Ibuprofen</b>	Motrin	Non-reactive
<b>Ibuprofen</b>	Proflex	Non-reactive
<b>Indomethacin</b>	Flexin continus	Non-reactive
<b>Indomethacin</b>	Indocid	Non-reactive
<b>Indomethacin</b>	Indomod	Non-reactive
<b>Interferon Alfa</b>	Roferon-A	Non-reactive
<b>Interferon Alfa</b>	Intron A	Non-reactive

<b>Interferon Alfa</b>	Viraferon	Non-reactive
<b>Interferon Alfa</b>	Wellferon	Non-reactive
<b>Isoniazide</b>	INH	Non-reactive
<b>Isosorbide Dinitrate</b>	Angitak	Non-reactive
<b>Isosorbide Dinitrate</b>	Cedocard	Non-reactive
<b>Isosorbide Dinitrate</b>	Isocard	Non-reactive
<b>Isosorbide Dinitrate</b>	Isoket	Non-reactive
<b>Isosorbide Dinitrate</b>	Isordil	Non-reactive
<b>Isosorbide Dinitrate</b>	Sorbichew	Non-reactive
<b>Isosorbide Dinitrate</b>	Sorbitrate	Non-reactive
<b>Isosorbide Dinitrate</b>	Sorbid SA	Non-reactive
<b>Ketoprofen</b>	Orudis	Non-reactive
<b>Ketoprofen</b>	Oruvail	Non-reactive
<b>Ketoprofen</b>	Powergel	Non-reactive
<b>Lactulose</b>	Lactugal	Non-reactive
<b>Lactulose</b>	Duphalec	Non-reactive
<b>Lansoprazole</b>	Heliclear	Non-reactive
<b>Lansoprazole</b>	Zoton	Non-reactive
<b>Lisinopril</b>	Carace	Non-reactive
<b>Lisinopril</b>	Zestril	Non-reactive
<b>Lisinopril</b>	Zestoretic	Non-reactive
<b>Lofexidine</b>	Britoflex	Non-reactive
<b>Loperamide</b>	Imodium	Non-reactive
<b>Loperamide</b>	Loperagen	Non-reactive
<b>Loperamide</b>	Norimide	Non-reactive
<b>Loratadine</b>	Claritin	Non-reactive
<b>Magnesium Alginate</b>	Gaviscon	Non-reactive
<b>Mannitol</b>	Gaviscon	Non-reactive
<b>Mefenamic Acid</b>	Ponstan	Non-reactive
<b>Mefenamic Acid</b>	Ponstel	Non-reactive
<b>Meperidine</b>	Demerol	Non-reactive
<b>Meperidine</b>	Pethidine	Non-reactive
<b>Metoclopramide</b>	Reglan	Non-reactive
<b>Methylphenidate</b>	Ritalin	Non-reactive
<b>Methylphenidate</b>	Concerta	Non-reactive
<b>Metronidazole</b>	Anabact	Non-reactive
<b>Metronidazole</b>	Elyzol	Non-reactive
<b>Metronidazole</b>	Flagyl	Non-reactive
<b>Metronidazole</b>	Metrogel	Non-reactive
<b>Metronidazole</b>	Metrolyl	Non-reactive
<b>Metronidazole</b>	Metrotop	Non-reactive
<b>Metronidazole</b>	Neuratop	Non-reactive
<b>Metronidazole</b>	Noritate	Non-reactive
<b>Metronidazole</b>	Rozex	Non-reactive
<b>Metronidazole</b>	Zidoval	Non-reactive

<b>Metronidazole</b>	Zymoet	Non-reactive
<b>Nadolol</b>	Corgard	Non-reactive
<b>Nadolol</b>	Corgaretic	Non-reactive
<b>Nadolol</b>	Corgartic	Non-reactive
<b>Naloxone</b>	Narcan	Non-reactive
<b>Naltrexone</b>	Antaxone	Non-reactive
<b>Naltrexone</b>	Nalorex	Non-reactive
<b>Naltrexone</b>	Trexan	Non-reactive
<b>Naproxen</b>	Aleve	Non-reactive
<b>Naproxen</b>	Condrotec	Non-reactive
<b>Naproxen</b>	Naprateg	Non-reactive
<b>Naproxen</b>	Naprosen	Non-reactive
<b>Naproxen</b>	Naprosyn	Non-reactive
<b>Naproxen</b>	Nycopren	Non-reactive
<b>Naproxen</b>	Synflex	Non-reactive
<b>Nicotine</b>	Nicoderm	Non-reactive
<b>Nicotine</b>	Nicorette	Non-reactive
<b>Nicotine</b>	Nicotinell	Non-reactive
<b>Nicotine</b>	Nicotrol	Non-reactive
<b>Nicotine</b>	Niquitin	Non-reactive
<b>Nifedipine</b>	Adalat	Non-reactive
<b>Nifedipine</b>	Nifecard	Non-reactive
<b>Nifedipine</b>	Nifar	Non-reactive
<b>Nifedipine</b>	Nif-ten	Non-reactive
<b>Nifedipine</b>	Procardia	Non-reactive
<b>Norethindrone</b>	BiNovum	Non-reactive
<b>Norethindrone</b>	Brevinor	Non-reactive
<b>Norethindrone</b>	Climagest	Non-reactive
<b>Norethindrone</b>	Climesse	Non-reactive
<b>Norethindrone</b>	Elleste Duet	Non-reactive
<b>Norethindrone</b>	Estracombi	Non-reactive
<b>Norethindrone</b>	Evorel	Non-reactive
<b>Norethindrone</b>	Kliofem	Non-reactive
<b>Norethindrone</b>	Kliovance	Non-reactive
<b>Norethindrone</b>	Loestrin	Non-reactive
<b>Norethindrone</b>	Micronor	Non-reactive
<b>Norethindrone</b>	Norlutin	Non-reactive
<b>Norethindrone</b>	Noriday	Non-reactive
<b>Norethindrone</b>	Norimin	Non-reactive
<b>Norethindrone</b>	Norinyl	Non-reactive
<b>Norethindrone</b>	Noristat	Non-reactive
<b>Norethindrone</b>	Nouvelle	Non-reactive
<b>Norethindrone</b>	Ovysmen	Non-reactive
<b>Norethindrone</b>	TriNovum	Non-reactive
<b>Norethindrone</b>	Trisquens	Non-reactive

<b>Norethindrone</b>	Utovian	Non-reactive
<b>Noscapine</b>	Narcotine	Non-reactive
<b>d/l-Octopamine</b>	No known trade names	Non-reactive
<b>Olanzapine</b>	Zyprexa	Non-reactive
<b>Orphenadrine</b>	Norflex	Non-reactive
<b>Orphenadrine</b>	Norgesic	Non-reactive
<b>Orphenadrine</b>	Disipil	Non-reactive
<b>Oxymetazoline</b>	Afrin	Non-reactive
<b>Oxymetazoline</b>	Neo-synephrine	Non-reactive
<b>Oxytetracycline</b>	Terra-Cortil	Non-reactive
<b>Oxytetracycline</b>	Terramycin	Non-reactive
<b>Oxytetracycline</b>	Trimovate	Non-reactive
<b>Paracetamol (Acetaminophen)</b>	Fortagesic	Non-reactive
<b>Paracetamol (Acetaminophen)</b>	Paradote	Non-reactive
<b>Paroxetine</b>	Paxil	Non-reactive
<b>Paroxetine</b>	Seroxat	Non-reactive
<b>Penicillin</b>	Combicillin	Non-reactive
<b>Penicillin</b>	Mefoxin	Non-reactive
<b>Pericyazine</b>	Amplan	Non-reactive
<b>Pericyazine</b>	Aolept	Non-reactive
<b>Pericyazine</b>	Apamin	Non-reactive
<b>Pericyazine</b>	Iryakin	Non-reactive
<b>Pericyazine</b>	Nemactil	Non-reactive
<b>Pericyazine</b>	Neulactil	Non-reactive
<b>Pericyazine</b>	Propetyl	Non-reactive
<b>Pericyazine</b>	Psycholept	Non-reactive
<b>Piperacillin</b>	Pipracil	Non-reactive
<b>Piperacillin</b>	Tazobactam	Non-reactive
<b>Prednisolone</b>	Delta-cortef	Non-reactive
<b>Prednisolone</b>	Econopred	Non-reactive
<b>Prednisolone</b>	Inflamase	Non-reactive
<b>Prednisolone</b>	Pediapred	Non-reactive
<b>Prednisolone</b>	Prelone	Non-reactive
<b>Progesterol</b>		Non-reactive
<b>Propranolol</b>	Beta-Progane	Non-reactive
<b>Propranolol</b>	Inderal	Non-reactive
<b>Propranolol</b>	Inderetic	Non-reactive
<b>Propranolol</b>	Inderex	Non-reactive
<b>Pseudoephedrine</b>	Afrinol	Non-reactive
<b>Pseudoephedrine</b>	Sudafed	Non-reactive
<b>Pseudoephedrine</b>	Tylenol Cold (combination)	Non-reactive
<b>Salbutamol</b>	Aerocrom	Non-reactive
<b>Salbutamol</b>	Aerolin	Non-reactive
<b>Salbutamol</b>	Airomir	Non-reactive
<b>Salbutamol</b>	Asmasal	Non-reactive

Salbutamol	Combivent	Non-reactive
Salbutamol	Duovent	Non-reactive
Salbutamol	Ventide	Non-reactive
Salbutamol	Ventodisks	Non-reactive
Salbutamol	Venotlin	Non-reactive
Salbutamol	Volmax	Non-reactive
Sennosides	Senokot	Non-reactive
Sodium Valproate	Depakene	Non-reactive
Sodium Valproate	Depakote	Non-reactive
Sodium Valproate	Epilim	Non-reactive
Testosterone	Andropatch	Non-reactive
Testosterone	Restandol	Non-reactive
Testosterone	Sustanon	Non-reactive
Testosterone	Virormone	Non-reactive
Testosterone	slang name "anabolic steroids"	Non-reactive
Thioridazine	Mellaril	Non-reactive
Thyroxine Sodium	Eltroxin	Non-reactive
Tramadol	Ultram	Non-reactive
Tramadol	Tramake	Non-reactive
Tramadol	Zamadol	Non-reactive
Tramadol	Zydol	Non-reactive
Trazodone	Desyrel	Non-reactive
Trazodone	Molipaxin	Non-reactive
Trazodone	Trialodine	Non-reactive
Warfarin Sodium	Coumadin	Non-reactive
Warfarin Sodium	Marevan	Non-reactive
Dexamphetamine Sulphate	Adderall	<b>Positive for Amphetamine (AMP)</b>
Dexamphetamine Sulphate	Adderall XR	<b>Positive for Amphetamine (AMP)</b>
Dexamphetamine Sulphate	Dexedrine	<b>Positive for Amphetamine (AMP)</b>
Methylenedioxyamphetamine (MDA)	Eve (slang)	<b>Positive for Amphetamine (AMP) and Ecstasy (MDMA)</b>
Methylenedioxyamphetamine (MDA)	Love Drug (slang)	<b>Positive for Amphetamine (AMP) and Ecstasy (MDMA)</b>
Allobarbitol	No known trade names	<b>Positive for Barbiturates (BAR)</b>
Alphenol	No known trade names	<b>Positive for Barbiturates (BAR)</b>
Amobarbital	Amytal	<b>Positive for Barbiturates (BAR)</b>
Amobarbital	Tuinal	<b>Positive for Barbiturates (BAR)</b>
Aprobarbital		<b>Positive for Barbiturates (BAR)</b>
Barbital		<b>Positive for Barbiturates (BAR)</b>
Butabarbital	Butisol	<b>Positive for Barbiturates (BAR)</b>
Butabarbital	Soneryl	<b>Positive for Barbiturates (BAR)</b>
Butalbital	Fioricet	<b>Positive for Barbiturates (BAR)</b>
Butalbital	Fiorinal	<b>Positive for Barbiturates (BAR)</b>
Butethal		<b>Positive for Barbiturates (BAR)</b>
Pentobarbital	Nembutal	<b>Positive for Barbiturates (BAR)</b>
Phenobarbitone (see also Phenobarbital)	Luminal	<b>Positive for Barbiturates (BAR)</b>
Phenobarbital	Donnatal	<b>Positive for Barbiturates (BAR)</b>

Secobarbital	Seconal	Positive for Barbiturates (BAR)
Alprazolam	Xanax	Positive for Benzodiazepines (BZO)
Bromazepam	Lexotan	Positive for Benzodiazepines (BZO)
Chlorazepate	Tranxene	Positive for Benzodiazepines (BZO)
Chlordiazepoxide	Librium	Positive for Benzodiazepines (BZO)
Clobazam	Frisium	Positive for Benzodiazepines (BZO)
Clonazepam	Clonopin	Positive for Benzodiazepines (BZO)
Clonazepam	Klonopin	Positive for Benzodiazepines (BZO)
Clonazepam	Rivotril	Positive for Benzodiazepines (BZO)
Delorazepam	Briantum	Positive for Benzodiazepines (BZO)
Diazepam	Diazemuls	Positive for Benzodiazepines (BZO)
Diazepam	Stesolid	Positive for Benzodiazepines (BZO)
Diazepam	Valclair	Positive for Benzodiazepines (BZO)
Diazepam	Valium	Positive for Benzodiazepines (BZO)
Estazolam	ProSom	Positive for Benzodiazepines (BZO)
Flunitrazepam	Rohypnol	Positive for Benzodiazepines (BZO)
Lorazepam	Ativan	Positive for Benzodiazepines (BZO)
Lormetazepam	Noctamide	Positive for Benzodiazepines (BZO)
Medazepam	Anxitol	Positive for Benzodiazepines (BZO)
Medazepam	Lerisum	Positive for Benzodiazepines (BZO)
Medazepam	Medacepan	Positive for Benzodiazepines (BZO)
Medazepam	Nobritol	Positive for Benzodiazepines (BZO)
Medazepam	Nobrium	Positive for Benzodiazepines (BZO)
Nitrazepam	Mogadon	Positive for Benzodiazepines (BZO)
Nitrazepam	Somnite	Positive for Benzodiazepines (BZO)
Oxazepam	Serax	Positive for Benzodiazepines (BZO)
Oxazepam	Ox-pam	Positive for Benzodiazepines (BZO)
Prazepam	Centrax	Positive for Benzodiazepines (BZO)
Prazepam	Demetrin	Positive for Benzodiazepines (BZO)
Temazepam	Restoril	Positive for Benzodiazepines (BZO)
Triazolam	Halcion	Positive for Benzodiazepines (BZO)
Buprenorphine	Subutex	Positive for Buprenorphine (BUP)
Buprenorphine	Temgesic	Positive for Buprenorphine (BUP)
Buprenorphine	Suboxone	Positive for Buprenorphine (BUP)
Efavirenz	Sustiva	Positive for Cannabinoids (THC)* * Urinary metabolite(s) only; parent compound is non-reactive
Doxylamine	Nyquil	Positive for Methadone (MTD)
Methadone Hydrochloride	Dolophine	Positive for Methadone (MTD)
Methadone Hydrochloride	Methadose	Positive for Methadone (MTD)
Methadone Hydrochloride	Physetone	Positive for Methadone (MTD)
d-Methamphetamine HCL	Desoxyn	Positive for Methamphetamine (mAMP)
d-Methamphetamine HCL	Methedrine	Positive for Methamphetamine (mAMP)
d-Methamphetamine HCL	Methamprex	Positive for Methamphetamine (mAMP)
l-Methamphetamine HCL	Vick's Inhaler	Positive for Methamphetamine (mAMP)
Methylenedioxymethamphetamine (MDMA)	Ecstasy (slang)	Positive for Methamphetamine (mAMP) and Ecstasy (MDMA)

Methylenedioxyamphetamine (MDMA)	XTC (slang)	Positive for Methamphetamine (mAMP) and Ecstasy (MDMA)
Methylenedioxyamphetamine (MDMA)	Adam (slang)	Positive for Methamphetamine (mAMP) and Ecstasy (MDMA)
Methylenedioxyamphetamine (MDMA)	E (slang)	Positive for Methamphetamine (mAMP) and Ecstasy (MDMA)
Ranitidine	Pylorid, Zantac	Positive for Methamphetamine (M-AMP)* *Urinary metabolite(s) only; parent compound is non-reactive
Morphine	Astramorph	Positive for Opiates (MOR, OPI)
Morphine	Cyclimorph	Positive for Opiates (MOR, OPI)
Morphine	Duramorph	Positive for Opiates (MOR, OPI)
Morphine	Morcap	Positive for Opiates (MOR, OPI)
Morphine	Morphine Sulfate	Positive for Opiates (MOR, OPI)
Morphine	MS Contin	Positive for Opiates (MOR, OPI)
Morphine	Oramorph	Positive for Opiates (MOR, OPI)
Morphine	Roxanol	Positive for Opiates (MOR, OPI)
Morphine	Severedol	Positive for Opiates (MOR, OPI)
Paracetamol (Acetaminophen) /Codeine Preparations	Tylenol 3	Positive for Opiates (MOR, OPI)
Paracetamol (Acetaminophen) /Codeine Preparations	Co-codamol	Positive for Opiates (MOR, OPI)
Paracetamol (Acetaminophen) /Codeine Preparations	Codafen	Positive for Opiates (MOR, OPI)
Paracetamol (Acetaminophen) /Codeine Preparations	Co-dydramol	Positive for Opiates (MOR, OPI)
Paracetamol (Acetaminophen) /Codeine Preparations	Kapake	Positive for Opiates (MOR, OPI)
Paracetamol (Acetaminophen) /Codeine Preparations	Remedine	Positive for Opiates (MOR, OPI)
Paracetamol (Acetaminophen) /Codeine Preparations	Solpadol	Positive for Opiates (MOR, OPI)
Paracetamol (Acetaminophen) /Codeine Preparations	Tylox	Positive for Opiates (MOR, OPI)
Pholcodine	Galenphol	Positive for Opiates (MOR, OPI)
Pholcodine	Strong BP	Positive for Opiates (MOR, OPI)
Pholcodine	Pavacol-D	Positive for Opiates (MOR, OPI)
Pholcodine	Thebacon	Positive for Opiates (MOR, OPI)
Procaine	Novocain	Positive for Opiates (OPI, MOP)
Acetaminophen with Codeine (see also Paracetamol with cod)	Tylenol 3	Positive for Opiates (OPI, MOR)
Acetaminophen with Codeine (see also Paracetamol with cod)	Tylenol with codeine	Positive for Opiates (OPI, MOR)
Codeine Phosphate	Codafen Continus	Positive for Opiates (OPI/MOR)
Codeine Phosphate	Codeine Linctus	Positive for Opiates (OPI/MOR)
Codeine Phosphate	Pediatric BP	Positive for Opiates (OPI/MOR)
Codeine Phosphate	Galcodine	Positive for Opiates (OPI/MOR)
Codeine Phosphate	Kapake	Positive for Opiates (OPI/MOR)
Codeine Phosphate	Migraleve	Positive for Opiates (OPI/MOR)
Codeine Phosphate	Solpadol	Positive for Opiates (OPI/MOR)
Codeine Phosphate	Tylox	Positive for Opiates (OPI/MOR)
Dihydrocodeine	DHC Continus	Positive for Opiates (OPI/MOR)
Dihydrocodeine	Paramol	Positive for Opiates (OPI/MOR)
Dihydrocodeine	Remedeine	Positive for Opiates (OPI/MOR)
Dihydrocodeine	Remedeine Forte	Positive for Opiates (OPI/MOR)
Ethylmorphine		Positive for Opiates (OPI/MOR)
Koalin and Morphine Mixture	Diocalm	Positive for Opiates (OPI/MOR)
Koalin and Morphine Mixture	Entersan	Positive for Opiates (OPI/MOR)
Koalin and Morphine Mixture	Opazimes	Positive for Opiates (OPI/MOR)

Hydrocodone	Lorcet	Positive for Opiates (OPI/MOR) and/or Oxycodone (OXY)
Hydrocodone	Loratab	Positive for Opiates (OPI/MOR) and/or Oxycodone (OXY)
Hydrocodone	Vicodin	Positive for Opiates (OPI/MOR) and/or Oxycodone (OXY)
Hydromorphone	Dilaudid	Positive for Opiates (OPI/MOR) and/or Oxycodone (OXY)
Hydromorphone	Hydrostat	Positive for Opiates (OPI/MOR) and/or Oxycodone (OXY)
Lamotrigine	Lamitcal	Positive for Phencyclidine (PCP)
Dextropropoxyphene	Darvon	Positive for Propoxyphene (PPX)
Dextropropoxyphene	Darvocet	Positive for Propoxyphene (PPX)
Dextropropoxyphene	Co-proximal	Positive for Propoxyphene (PPX)
Amitriptyline	Elavil	Positive for Tricyclic Antidepressants (TCA)
Amitriptyline	Lentizol	Positive for Tricyclic Antidepressants (TCA)
Amitriptyline	Tryptizol	Positive for Tricyclic Antidepressants (TCA)
Amitriptyline	Triptafen	Positive for Tricyclic Antidepressants (TCA)
Amitriptyline	Triptafen-M	Positive for Tricyclic Antidepressants (TCA)
Doxepin	Sinequan	Positive for Tricyclic Antidepressants (TCA)
Doxepin	Xepin	Positive for Tricyclic Antidepressants (TCA)
Lofepramine	Gamanil	Positive for Tricyclic Antidepressants (TCA)
Lofepramine	Lomont	Positive for Tricyclic Antidepressants (TCA)
Promethazine	Allerfen	Positive for Tricyclic Antidepressants (TCA)
Promethazine	Anergan	Positive for Tricyclic Antidepressants (TCA)
Promethazine	Antinaus	Positive for Tricyclic Antidepressants (TCA)
Promethazine	Aprobit	Positive for Tricyclic Antidepressants (TCA)
Promethazine	Avomine	Positive for Tricyclic Antidepressants (TCA)
Promethazine	Baymethzine	Positive for Tricyclic Antidepressants (TCA)
Promethazine	Diprozin	Positive for Tricyclic Antidepressants (TCA)
Promethazine	Fenergan	Positive for Tricyclic Antidepressants (TCA)
Promethazine	Methazine	Positive for Tricyclic Antidepressants (TCA)
Promethazine	Phenergen	Positive for Tricyclic Antidepressants (TCA)
Phenytoin	Dilantin	Possible Positive for Barbiturates (BAR) * Urinary metabolite(s) only; parent compound is non-reactive
Phenytoin	Epanutin	Possible Positive for Barbiturates (BAR) * Urinary metabolite(s) only; parent compound is non-reactive
Phenytoin	Epitard	Possible Positive for Barbiturates (BAR) * Urinary metabolite(s) only; parent compound is non-reactive
Sertraline	Zoloft	Potential Positive for Benzodiazepines (BZO)
Venlafaxine	Effexor	Potential Positive for Phencyclidine (PCP)
Venlafaxine	Effexor XL	Potential Positive for Phencyclidine (PCP)